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**Protac Incontinence Cover**

GB - Directions for use

**Protac Inkontinenzbezug**

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# GB - Directions for use

Protac Incontinence Cover

Protac Incontinence Cover with MESH

You have selected a quality Protac product. Protac guarantees that you have purchased an original, Danish designed product developed in consultation with experts.

Protac has developed, produced and sold sensory-stimulating aids to the healthcare sector in Denmark, the rest of Europe and Australia since 1994.

Protac products are developed on the basis of theories on sensory integration, and are supported by extensive research, documentation and case studies.

Protac focuses strongly on protection of the environment and safety during its production process, and the fabric used for our Incontinence Cover is OEKO-certified.

For more details, go to [www.protac.dk](http://www.protac.dk)

GB

## Labelling

Our products are labelled according to legislation concerning medical equipment.

The label with details of manufacturer, raw materials and washing instructions is sewn inside the cover to avoid irritating the user. This location also protects the label from wear and tear.

Key to label symbols: please refer to **Key to symbols**

## General precautions - Safety

- ⚠ • We recommend that the opening of the incontinence cover is at the opposite end to that of the duvet cover (**not applicable for incontinence cover with mesh top**)
- ⚠ • The same risk precautions apply as for the use of a plastic bag:
  - A Protac Ball Blanket™ used with an incontinence cover **must not** be used for children without supervision.
  - A Protac Ball Blanket™ with incontinence cover **must not** be used for patients unable to care for themselves without supervision.
  - A Protac Ball Blanket™ used with an incontinence cover **must not** be pulled over the head.
- ⚠ • We recommend using the incontinence cover version with a mesh top (a breathable, net panel) for users unable to care for themselves.
- ⚠ • **Must not** be used near an open flame.

## Product purpose and areas of use

The Protac Incontinence Cover was developed to help sufferers of function impairment.

Children, young and old people can benefit from the cover.

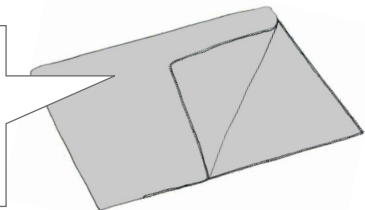
- Brain damage, including spasticity, Huntington's Chorea, Parkinson's Disease and other ailments
- ADHD and autism
- Psychiatric diseases, such as dementia, schizophrenia, depression, anxiety, anorexia etc.
- Development problems, sensory problems, deaf/blind, etc.
- Stress

The Protac Incontinence Cover is used to protect the Protac Ball Blanket™ or other duvets in the event of patient incontinence.

## Using the product

The cover is intended to fit over the Protac Ball Blanket™. An ordinary duvet cover should be used over this.

The cover is easy to fit, with zips on two sides.

GB

## Product service life

The product has an expected service life of 5 years.

This is based on correct use, cleaning, washing, service and maintenance.

Products used beyond the aforementioned service life should be assessed for further use.

Responsibility for using the product beyond the stated service life rests with the user.

## Cleaning

All Protac Incontinence Covers can be washed. The regularity and need for washing depends on use. See below for washing and cleaning instructions.

## Protac Incontinence Cover

- Incontinence Cover, item no. 300-302 and 300-304 can be washed in a min. 6 kg capacity washing machine with washing powder with no bleaching agents - see also the washing instructions in the cover.

⚠ • Spot remover **should not** be used to avoid damage to the PU coating.

Protac cannot be held liable for incorrect washing and maintenance.

## Maintenance

We recommend washing the cover regularly. Use of a laundry bag is recommended.

## Transport and storage

The Protac Incontinence Cover is supplied in a sealed plastic bag.

The cover is ready for use.

If you are not using the cover, store it carefully in its bag to protect from dirt and damp.

## Disposal

The cover can be disposed of as combustible refuse.

Always dispose of refuse according to local rules.

## Guarantee

2 year guarantee against manufacturing defects.

### The guarantee will be invalid in the event of the following:

- over-use
- misuse
- non-maintenance or incorrect maintenance

**Traceability:** The month and year of manufacturer are marked on the washing instructions label.

## Key to symbols

Example only. Always follow the recommended care instructions on the label found inside the product

GB

**Washing bowl:** Maximum washing temperature (see care label)

**Warning:**  
See under **Safety**

**Triangle:**  
Do not bleach

User information

**Square with a circle:**  
tumble drying possible  
(see care label)

For indoor use

**Iron:**  
Do not iron

Manufacturer

**Circle:**  
Do not dry clean

Product serial number (see care label)

## Technical specifications

### Dimensions, Protac Incontinence Cover:

Protac Incontinence Cover, Adult:	Width: 40 cm	Length: 200/220 cm
Protac Incontinence Cover with MESH panel on top, adult:	Width: 140 cm	Length: 200/220 cm
Protac Incontinence Cover, Junior:	Width: 100 cm	Length: 140 cm

**Standards:** Comply with general requirements for aids acc. to the following harmonized standards:

**Clinical evaluation:** EN 12182:2012

**Risk analysis:** DS/EN ISO 14971

### Fire test:

ISO 12952 part 1 & 2 : 2010 Style 300-302 only.

Model 300-304 is made of polyester with polyurethane coating and polyester MESH net on the top - this model is **not** flame-proof.

## Guide

- to daily inspection of the Protact Incontinence Cover before use

No.	Check	Remarks
1	<ul style="list-style-type: none"><li>All parts are in place, no loose parts</li></ul>	
2	<ul style="list-style-type: none"><li>No sign of holes, damage to the textile or seams</li></ul>	

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### Protac A/S